

## Philips FRx AED/Defibrillator Verifying Readiness



### Step two:

Check for green flashing readiness light (flashes once every 5 seconds). If not flashing, battery may need replacing.

### Step three:

Press and release information button to check battery and readiness status of device. If no vocal prompts are heard, replace battery.

### Step one:

Check expiry dates of pads and replace if expired.

### Step four:

Check expiry date of spare pads; replace if expired.  
Check install before date of spare battery; replace if past install before date  
Check device is clean; use Clinell Universal Wipes to clean device.

### Contact:

For further information, email the Trust Resuscitation Lead; [elft.resuscitation@nhs.net](mailto:elft.resuscitation@nhs.net) or call, 020 8223 8022 (office), 07392 285 430 (mobile)

### Ordering consumables:

Please order replacement pad(s) and battery(s) directly from the ELFT PPE Warehouse via the PPE Platform: [www.elftwarehouse.co.uk](http://www.elftwarehouse.co.uk)