

Philips HS1 AED/Defibrillator Verifying Readiness / Daily Checks



Step two:
Check for green flashing readiness light (flashes once every 5 seconds). If not flashing, battery may need replacing.

Step three:
Press and release information button to check battery and readiness status of device. If no vocal prompts are heard, replace battery.

Step one:
Check expiry dates of pads and replace if expired.

Step four:
Check expiry date of spare pads; replace if expired.
Check install before date of spare battery; replace if past install before date
Check device is clean; use Clinell Universal Wipes to clean device.

Contact:
For further information, email the Trust Resuscitation Lead; elft.resuscitation@nhs.net or call, 020 8223 8022 (office), 07392 285 430 (mobile)

Ordering consumables:
Please order replacement pad(s) and battery(s) directly from the ELFT PPE Warehouse via the PPE Platform: www.elftwarehouse.co.uk